## Lambda Light Chains Positive Control L

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IVD

#### INTENDED USE

Solution to be used for the accuracy control of the data obtained for lambda light chains assays, in manual and automated methods.

#### REAGENT – 1 vial containing 1 ml.

Ready to use liquid reagent containing known concentration of lambda light chains.

#### STORAGE AND STABILITY

The solution is stable up to the expiry date stated on the label, if stored at 2-8 °C.

Do not use the control after the expiry date.

Keep the control tightly closed to prevent contaminations during its use.

#### ASSIGNED VALUES

Concentration value and acceptable range are stated on the label and vary from lot to lot.

Calibrated against Beckman Coulter Proteins Calibrator.

#### PRECAUTIONS IN USE

All reagents contain inactive components such as preservatives (Sodium azide or others), surfactants etc. The total concentration of these components is lower than the limits reported by 67/548/ECC and 88/379/EEC directives about classification, packaging and labelling of dangerous substances. However, the reagents should be handled with caution, avoiding swallowing and contact with skin, eyes and mucous membranes.

The use of the laboratory reagents according to good laboratory practice is recommended.

#### Waste Management

Please refer to local legal requirements.

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