Microalbumin Turbidimetric Control L Controllo Microalbumina Turbidimetrico L

REF

GD8438 00

IVD

INTENDED USE

Solution of human albumin suitable for the accuracy and precision control of the turbidimetry methods in clinical laboratories.

REAGENT – 2 vials containing 2 ml. Ready to use.
Human albumin. Sodium azide 0.95 g/l.

STORAGE AND STABILITY

The solution is stable up to the expiry date stated on the label, if stored at 2-8 °C. Do not use the control after the expiry date.

Keep the control tightly closed to prevent contaminations during its use.

Presence of particles and turbidity is sign of deterioration.
Opened vials are stable for 30 days at 2-8 °C.

Note

Bacterial contamination may cause reduction in the stability of the control.

INSTRUCTION FOR USE

Use the control as a patient sample following the instruction of the corresponding analytical method.

ASSIGNED VALUES

The concentration value and limits have been assigned using the turbidimetric method. However, the results may slightly differ from these values because of the different methods and instruments used in different laboratories. It is useful that each laboratory establishes its own precision parameters.

PRECAUTIONS IN USE

All reagents contain inactive components such as preservatives (Sodium azide or others), surfactants etc. The total concentration of these components is lower than the limits reported by 67/548/ECC and 88/379/EEC directives about classification, packaging and labelling of dangerous substances. However, the reagents should be handled with caution, avoiding swallowing and contact with skin, eyes and mucous membranes.

The use of the laboratory reagents according to good laboratory practice is recommended.

The reagents from human donors have given negative results to anti-HIV 1/2, HBsAg and anti-HCV. It is recommended to handle with caution.

Waste Management

Please refer to local legal requirements.

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