PLASMAPROTEIN PATHOLOGICAL CONTROL L CONTROLLO PATOLOGICO PLASMAPROTEINE L

REF

GD8464 00

IVD

INTENDED USE

Human serum containing different concentrations of plasma proteins suitable for the accuracy and precision control of the analytical methods in clinical laboratories.

REAGENT – 2 vials containing 2 ml. Ready to use.
Human serum. Sodium azide 0.95 g/l.

STORAGE AND STABILITY

The solution is stable up to the expiry date stated on the label, if stored at 2-8 °C. Do not use the control after the expiry date. Keep the control tightly closed to prevent contaminations during its use.

Presence of particles and turbidity is sign of deterioration.
Opened vials are stable for 30 days at 2-8 °C.

Note

Bacterial contamination may cause reductions in the stability of many components of the control.

ASSIGNED VALUES

The concentration values and limits for each parameter have been assigned using the method specified in the attached table. However, the results may slightly differ from these values because of the different methods and instruments used in different laboratories. It is useful that each laboratory establishes its own precision parameters.

PRECAUTIONS IN USE

All reagents contain inactive components such as preservatives (Sodium azide or others), surfactants etc. The total concentration of these components is lower than the limits reported by 67/548/ECC and 88/379/EEC directives about classification, packaging and labelling of dangerous substances. However, the reagents should be handled with caution, avoiding swallowing and contact with skin, eyes and mucous membranes.

The use of the laboratory reagents according to good laboratory practice is recommended.

The reagents from human donors have given negative results to anti-HIV 1/2, HBsAg and anti-HCV. It is recommended to handle with caution.

Waste Management

Please refer to local legal requirements.

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AMS S.p.A. – Registered Office and Plant